

## SPECIFICATION

Product Name	Shoulder Support Brace with Hot & Cold Gel Pack
Product Model	ZC-SHO-11063
Material	Neoprene + Velcro
Included Components	Shoulder Support Brace, Reusable Gel Pack, Manual
Shoulder Brace Diameter	10in(L) x 9in(W) x 0.79in(H) 26cm(L) x 23cm(W) x 2cm(H)
Shoulder Brace Weight	5.33 oz   151 g
Gel Pack Weight	5.68 oz   161 g

## DESCRIPTION

Provides stability and compression for shoulder injuries like sprains, dislocations, and rotator cuff tears. Enhances blood flow, limits unwanted movement, and supports faster recovery.

# Zeencare SHOULDER SUPPORT BRACE WITH HOT & COLD GEL PACK



# GEL PACK FOR HOT & COLD THERAPY



## REUSABLE GEL PACK

- Microwave for hot therapy
- Freeze for cold therapy

# 4 STEPS FOR EASY USE



1 Connect the strap to the buckle and secure the lower Velcro on your shoulder.



2 Pull the long strap through the clip, adjust to fit, and fasten the Velcro.



3 Adjust and secure the two top Velcro straps for support.



4 Wrap the bottom Velcro around your arm and fasten it in place.